
# AVENT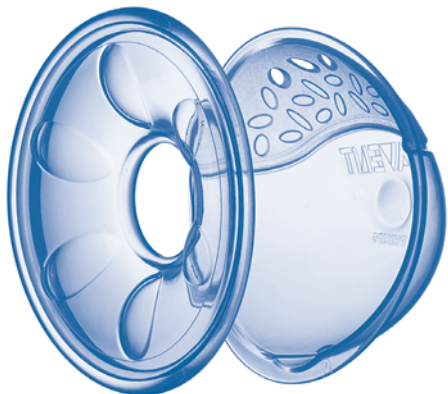

## Philips AVENT ISIS Breast Pump

- FR Tire-lait ISIS
- DE ISIS-Milchpumpe
- NL ISIS Borstkolf
- ES Extractor de leche ISIS
- IT Tiralatte ISIS
- PT Bomba Tira Leite ISIS
- NO ISIS Brystpumpe
- SE ISIS Bröstpump
- PL Laktator ISIS
- RU ISIS Молокоотсос

## AVENT Nipple Protectors

- Protège-mamelons Tendresse
- Brusthütchen - Schmetterlingsform
- Tepelbeschermers
- Protege Pezones
- Paracapezzoli in silicone
- Protectores de mamilo
- Brystskjold
- Bröstvårtsskydd
- Silikonowe osłonki piersi
- Защитные накладки на сосок



Philips of Holland. All logos brands and product names are trademarks or registered trademarks of their respective holders and are hereby recognised as such.

AVENT, Suffolk, CO10 7QS, England  
[www.philips.com/AVENT](http://www.philips.com/AVENT)

421335410090

CE

PHILIPS

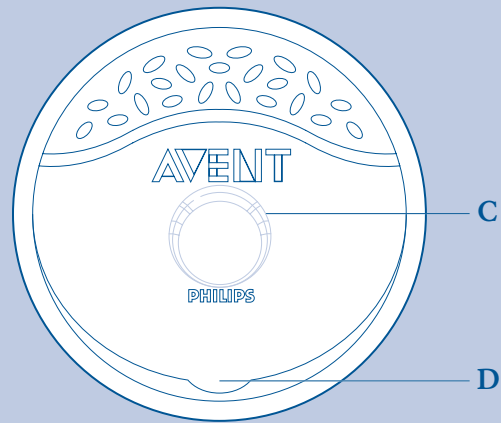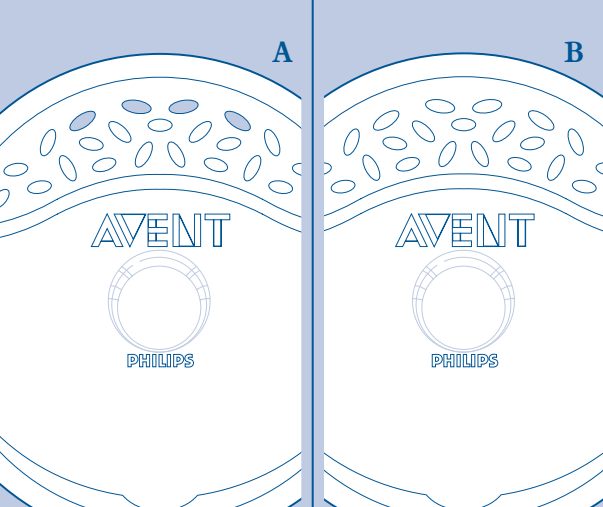GB

## AVENT ISIS Comfort Breast Shells with Ultrasoft Backing Cushions

Contents: 2 Ventilated Breast Shells, 2 Non-ventilated Breast Milk Saver Shells (**NO HOLES**) and 2 Ultrasoft Backing Cushions

- A. Ventilated Shells** – to protect sore or cracked nipples
- B. Non-ventilated Breast Milk Saver Shells** – to collect leaking breast milk
- C. Flat area** – for stability
- D. Pouring spout** – for accurate pouring

The AVENT ISIS Comfort Breast Shells are worn inside the bra to help you overcome some common problems you may encounter during breastfeeding.

### To protect sore or cracked nipples and help relieve engorgement

- ◆ Attach the Ultrasoft Backing Cushions to the Ventilated Shells and slip them inside your bra. The soft cushions will mould comfortably to the shape of your breasts and the domed shells protect your sore or cracked nipples from chafing. The ventilation holes allow the air to circulate.
- ◆ The Ventilated Shells can also be used to help relieve engorgement. They exert a gentle pressure around the areola and induce a slow leaking of your milk. **DO NOT** feed this milk to your baby, as milk collected over a long period of time may become contaminated.

### To collect breast milk for later use

- ◆ Sterilise the Non-ventilated Breast Milk Saver Shells (no holes) and the silicone Backing Cushions in a Philips AVENT Steam Steriliser (Electric, Electronic or Microwave) or by boiling for 5 minutes.
- ◆ Attach the Ultrasoft Backing Cushions to the Breast Milk Saver Shells. While you are feeding or expressing from one side, slip an assembled Breast Milk Saver Shell inside your bra on the other side to collect any leaking milk. Only milk collected whilst feeding or expressing should be stored for later use.

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◆ To empty the shell, carefully remove the silicone Backing Cushion and pour the breast milk into a sterilised bottle or VIA Cup. Place in a fridge or freezer as soon as possible after collection. Breast milk can be stored for up to 48 hours in the fridge (not in the door) or 3 months in the freezer.

◆ Wash all items thoroughly after use.

**NB:** It is recommended not to wear breast shells for longer than 40 minutes at a time as they may cause undue pressure on the milk ducts.

## Other items in the Philips AVENT Breastfeeding Range

◆ To relieve engorgement immediately prior to feeding, you can also use the **Philips AVENT ISIS Breast Pump** to express a small amount of milk to soften the breast and allow your baby to latch on.

◆ To allow you to continue breastfeeding without pain, even with sore or cracked nipples, use the **AVENT Nipple Protectors**. The soft, odourless, ultra fine silicone protects the nipple whilst the unique cut away shape still allows skin to skin contact with your baby.

◆ If you have flat or inverted nipples, use the **AVENT Nipplette™** for a few minutes just before feeding, to pull the nipple out and allow your baby to latch on. The AVENT Nipplette can also be used to effect a permanent cure after you have stopped breastfeeding.

For further information on the complete Philips AVENT range of bottle and breastfeeding products:

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